

INSTALLATION INSTRUCTIONS – MSL SERIES 1 MOBILE TROLLEY

The MSL Series 1 Mobile Trolley is designed and manufactured with a view to assist installation. The light is supplied as a number of pre-assembled components. When installing please follow the instructions in the detailed sequence. (1-5)

1. Fit Wheels to Base. (B)

Note: The two braked wheels are to be mounted opposing each other for increased stability.

2. Assemble Post to Base. (A & B)

- Remove M6 x 30mm bolt, washer and retaining disc from post. (A)
- Fit post (A) into base. (B)
- Secure bolt and washer using a 10mm spanner.

3. Install Light Head. (D)

- Remove 1 x M4 screw and washer from light head arm insert.
- Plug in low voltage cables using Molex connectors and slide back into lower arm. (C) Align screw holes in lower arm (C) and install screw and washer using 2.5mm Hex key.

Note: Screw must be securely tightened.

4. Install Counterweight. (E)

- Remove 1 x M5 screw, slide counterweight (E) into pivot block (F) and secure using a 3mm Hex key.

5. Power up unit to test.

