

# INSTALLATION INSTRUCTIONS – MSL SERIES 3 MOBILE TROLLEY

The MSL Series 3 Wall Mount is designed and manufactured with a view to assist installation. The light is supplied as a number of pre-assembled components. When installing please follow the instructions in the detailed sequence. **(1-8)**

## 1. Fit Wheels to Base. (B)

**Note:** The two braked wheels are to be mounted opposing each other for increased stability.

## 2. Assemble Post to Base. (A & B)

- Remove M6 x 30mm bolt, washer and retaining disc from post. **(A)**
- Fit post **(A)** into base **(B)**
- Secure bolt and washer using 10mm spanner. (Not supplied)

## 3. Install Light Head. (D)

- Remove 1 x M6 screw and washer from light head arm insert, Plug in low voltage cables using Molex connectors and carefully push back into arm. **(C)**
  - Align screw hole in arm and head insert and secure screw and washer using a 4mm Hex Key.
- Note:** Screw must be securely tightened.
- Fit Sterilisable handle.

## 4. Install Counterweight. (E)

- Remove 1 x M6 screw, slide counterweight **(E)** into pivot block **(F)** and secure using a 4mm Hex key.

## 5. Power up unit to test.

