

# INSTALLATION INSTRUCTIONS MSL SERIES 3 MOBILE TROLLEY

The MSL Series 3 Mobile Trolley is designed and manufactured with a view to assist installation. The light is supplied as a number of pre-assembled components. When installing follow the instructions in the detailed sequence (1-5).

1. **Fit Wheels to Base. (B)**

**Note:** The two braked wheels are to be mounted opposing each other for increased stability.

2. **Assemble Post to Base. (A & B)**

- Remove M6 x 30 bolt, washer and retaining disc from post (A).
- Fit post (A) into base (B).
- Secure bolt and washer using a 10mm spanner.

3. **Install Light Head. (D)**

- Remove 1 x M6 screw and washer.
- Plug in low voltage cables. **Note:** for LED models check cable colour coding for correct polarity. Carefully push connectors back in arm (C).
- Align screw holes, slide into arm (C), install screw and washer using a 4mm Hex key.  
**Note:** Screw must be securely tightened.
- Fit sterilisable handle.

4. **Install Counterweight. (E)**

Remove 1 x M6 screw, slide counterweight (E) into pivot block (F) and secure using a 4mm Hex key.

5. **Power up unit to test.**

